

Instruction for Use



Oxygen Mask Kit

Intended Use

This oxygen mask kit is intended to replace the nasal cannula supplied with a Firesafe™ Oxygen Therapy Kit when the patient prefers to use an oxygen mask. This kit includes a medium concentration adult oxygen mask and short oxygen tube.

Warnings!

1. Keep this oxygen mask kit free from oil and/or grease.
2. Never administer oxygen therapy while smoking or near an open flame.
3. This kit is intended for single patient use and should not be used with more than one patient due to the risk of cross-contamination.
4. Keep the tubing straight and free of kinks.
5. Do not sterilise this oxygen mask kit.
6. Do not immerse this oxygen mask kit in any fluid or allow fluid to enter the gas pathways.

Cleaning

Clean the outside of the tubing and the mask with a clean damp cloth. Do not use strong detergents, alcohol or scented lotions or soaps.

Replacement and Disposal

It is recommended to replace the oxygen mask if it becomes stiff, discoloured or uncomfortable.

Device Specification


Suitable age group	Adult
Mask Type	Simple, medium concentration
Operating Temperature	5 to 40 °C
Transit & Storage Temperature range	-20 to 50 °C

Kit Contents

This kit contains the following components:

1 x medium concentration adult oxygen mask - Salter Labs part number 8115

1 x oxygen tube, 0.3 metres long - Salter Labs part number 2001-1

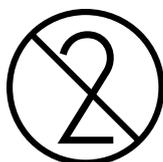

This kit is for
single patient use



No smoking



No naked flames

Using the Oxygen Mask Kit

1. Remove the nasal cannula from the Firesafe™ Oxygen Therapy Kit.
2. Connect the free end of the Oxygen Mask Kit to the free end of the Firesafe™ Oxygen Therapy Kit.
3. Place the oxygen mask over the nose and mouth, securing it to the head with the elastic strap.

